Nipple Formers

Bruksanvisning

Vårtformare

Bruksanvisning

Brystvorteformere

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Bruksanvisning

Brystknoppeformer

EN

Nipple Formers

The Medela Nipple Formers are comfortable to wear. The discreet shape and the flexible silicone membrane, which is gentle on the skin, mean that they are pleasant and invisible to wear. The ventilation holes in the top of the shells allow air to circulate, thus preventing humidity and promoting healing. The Nipple Former prepares flat or inverted nipples for breast-feeding. Flat or inverted nipples can make breastfeeding difficult. Some babies have troubles latching on correctly to these nipples, causing sore nipples. Gentle pressure on the areola from the silicone membrane gradually forces the nipple out. The therapy can be started during pregnancy (from about 32nd week onwards). Start wearing the Nipple Former for one hour a day and gradually increase to approximately 8 hours daily and also after birth between feedings (1/2–1 hour before breastfeeding). Avoid wearing at night to prevent tender spots.

Assembly and Use

Put the silicone membrane on the shell. Be sure it snaps on all the way around. Place the assembled Nipple Former inside your bra with the opening in the silicone membrane centered over your nipple. The ventilation holes in the shell must point upwards. We recommend wearing a nursing bra when using or a former; nursing bras are roomier and the former will fit more comfortably.
Caution: Do not feed the breast milk collected in the shell to the baby. Discard this milk.

Cleaning Instructions

Before first use and subsequently once daily

Place the separated parts in a small pan filled with cold water. Distilled water is recommended – if tap water is used, a teaspoon of citric acid may be added to the water to avoid a build-up of limescale. Bring the water to boil and allow it to continue boiling for approximately 5 minutes. Drain off the water and place the parts on a clean cloth to dry.

After each use

Separate the parts. Shells and membranes should be washed with a mild cleaning solution, rinsed thoroughly in cold water and dried on a clean cloth.

Note

Shells and membranes can be put in a dishwasher or autoclave.

Material

Shell: Polypropylene / Silicone membrane: Silicone

Disposal

According to local regulations. CE-mark according to 93/42/EEC

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